

# Typical Results

Before



Courtesy of Mark Rubin MD, USA

After 3 treatments



Before



Courtesy of Mark Rubin MD, USA

After 5 treatments



Before



Courtesy of Steven D. Shapiro MD, USA

After 5 treatments



To start your own  
before and after journey ask  
about Legend M<sup>2</sup>™  
skin treatment today



## Legend Pro

Dare to imagine

Treat your skin with  
**Legend M<sup>2</sup>™**



[legendskin.com](http://legendskin.com)

## Why Legend M<sup>2</sup>™?

Now you can treat the causes of skin aging and not just the symptoms

- ✓ Targets multiple skin layers to address the causes of aging skin
- ✓ Immediately visible results and long term efficacy
- ✓ A short treatment with minimal pain and downtime

## What is it?

### The Lumenis Legend M<sup>2</sup>™ combination therapy

Is a non-surgical treatment technique that reaches multiple layers of the skin and combines a wide range of solutions from skin resurfacing and regeneration to wrinkle reduction.

By incorporating two leading, clinically proven technologies, Legend M<sup>2</sup>™ provides immediate results that are maintained in the long term, with minimal pain and almost no downtime.

### Is It right for me?

Lumenis energy-based treatments are not suitable for everyone and do carry some risks.

Legend M<sup>2</sup>™ treatment is not suitable for patients with pacemakers, defibrillators, or any implanted electronic devices, as well as metal implants in the treatment area. Although very rare, risks may include damage to natural skin texture, skin redness, swelling, bruising and itching. Be sure to consult with your treatment provider before choosing this treatment.

## What should I expect?

### When will I see results?

You will see and feel the natural looking effect from your very first treatment. In order to get the best results, a course of 3-5 treatments every 4-6 weeks is highly recommended. A series of treatments will ensure enhanced results that stay with you longer.

### What can I expect after treatment?

After Legend M<sup>2</sup>™ treatments, you will immediately see an improved facial appearance. You may experience some redness or swelling, depending on the course of treatment created for you by your practitioner. This may last a couple of hours and up to a few days. In the following days, you may see and feel minor scabbing that will naturally heal and shed off. You may cover the area with some make-up. Each Legend M<sup>2</sup>™ treatment takes about 30 minutes. Immediately after the treatment, you can return to your daily activities.

## How does it work?


### Legend M<sup>2</sup>™ treatment defeats the signs of aging by combining the two M's:

- ✓ **Macro** volumetric heating of the tissue with TriPollar® RF
- ✓ Precise **Micro** heating of focal points with VoluDerm™ RF-assisted ultra-thin electrodes

Legend M<sup>2</sup>™ long-term results accomplished by the **macro** and **micro** thermal effect leads to the regeneration of collagen and the replenishing of the facial skin.

### Legend M<sup>2</sup>™ treatment offers the combination of these two technologies, working together to ensure your satisfaction.

**TriPollar® RF** triggers regeneration of collagen and elastin fibers by applying controlled heating of the deep dermis. The contraction of collagen fibers creates the effect over the skin. The effect is noticeable immediately after treatment.

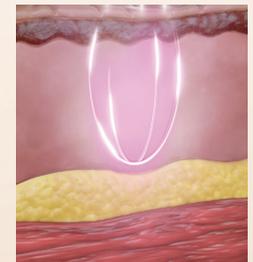

**VoluDerm™** encourages the generation of new and healthy skin by applying RF-assisted ultra-thin electrodes, creating controlled micro wounds that trigger the production of new collagen, elastin and hyaluronic acid.

